

Nasolabial fold & Pores






Before


2months follow up After 4th treatment

Parameter

1st pass: Q 4.5-1.0J (Spacing 1.5mm, Length 25mm) 2nd pass: Q 3.0-0.7J (Spacing 1.5mm, Length 25mm)

Fine wrinkles

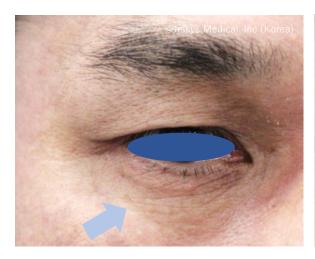

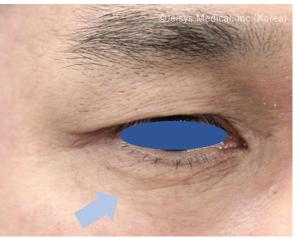

Before

1st treatment after

Parameter

2.0mm cartridge

0.2 level, 1.5mm spacing, 30 shots, 3 passes

Nasolabial fold





Before

Immediate result


Parameter

1st pass : Q 4.5-1.0J(Spacing 1.5mm, Length 25mm) 2nd pass : Q 3.0-0.7J(Spacing 1.5mm, Length 25mm)

2 weeks intervals, total 4 sessions treatment (Shot count: 300shot/session)





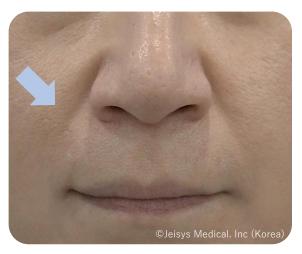

2months follow up After 4th treatment

Parameter

 $1st\ pass: Q\ 4.5-1.0J(Spacing\ 1.5mm,\ Length\ 25mm)\\ 2nd\ pass: Q\ 3.0-0.7J(Spacing\ 1.5mm,\ Length\ 25mm)\\$

2 weeks intervals, total 4 sessions treatment (Shot count: 300shot/session)

Nasolabial fold & Cheek





Before Immediate result

Parameter

 $1st\ pass: Q\ 4.5-0.9J (Spacing\ 1.5mm,\ Length\ 25mm) \\ 2nd\ pass: Q\ 3.0-0.8J (Spacing\ 1.5mm,\ Length\ 25mm)$

Face Line





Before

2months after

Parameter

Total: 400shots Q 4.5: 1.3J, Q 3.0: 0.8 \sim 1.0J, Q 1.5: 0.3J

Cheek





Before

4months after

Total: 400shots Q 4.5: 1.3J, Q 3.0: 0.8 \sim 1.0J, Q 1.5: 0.3J



Before




1st treatment after

Total: 400shots Q 4.5: 1.0J, Q 3.0: 0.8J, Q 1.5: 0.2J

(Spacing 1.3mm)

ALL FACE



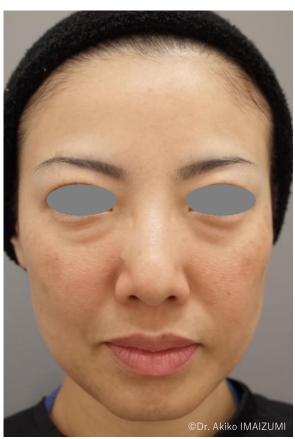

after Before

Parameter

Total: 350shots Q 4.5: 1.0J, Q 3.0: 0.8J, Q 1.5: 0.5J

The parameter indicated above is only for reference. Please note that reactions may vary according to patient's skin type

Cheek / Nasolabial fold / Jew line





Before after





Before after

Parameter

Total: 350shots

Q 4.5: 1.0J, Q 3.0: 0.8J, Q 1.5: 0.5J

ALL FACE





Before

1month after





Before

1month after

Parameter

Total: 350shots

Q 4.5: 1.0J, Q 3.0: 0.8J, Q 1.5: 0.5J